

PHILIPS
RESPIRONICS

Wisp

Pediatric nasal mask

Rediscover dreams

Leak

Correction Dial

For less intrusive
mask adjustment

Click fit headgear clips

with adjustable straps for
quick and easy adjustments.

Clips 1104974

Optional tube wrap

with tubing
management loop

Tubing w/ cover subassembly 1104977

Headgear with crown strap

Headgear 1104973

Wisp Pediatric nasal mask

Fit Pack with
Exhalation Port 1104961

Fit Pack includes 3 cushion sizes (SC Small,
SC Medium, SC Large), 1 frame and 1 headgear.

Cushion design

Featuring auto seal groove
technology, delivering
minimal contact.

SC Small 1104969

SC Medium 1104970

SC Large 1104971

360° rotating elbow

gives the child the
freedom to move

Soft fabric frame

Frame 1104972

SE Elbow 1104976

Helpful tips

For more fitting tips, find out how to adjust for leaks or cleaning instructions, please take a look at our instructional videos, visit: www.sleepapnea.com/products/masks/



- Before using the mask, hand wash the entire mask before use (see Mask Care in the Instructions for use). Wash the child's face. Do not use moisturizer or lotion on the child's face.
- Three cushion sizes (SC Small, SC Medium, SC Large) are available. Place the cushion-sizing gauge over the child's nose to determine the size.
- The most common mistake when fitting is over-tightening the headgear. Signs of overtightening include redness, sores, or bulging skin around the edges of the mask.

Fitting instructions

Achieving the right fit with Wisp pediatric is easy. Simply follow the instructions below. Firstly, select the appropriate cushion size. There are three cushion sizes available:



SCS (small)



SCM (medium)



SCL (large)



Use the sizing gauge to select the cushion that fits the width of the child's nose without blocking the nostrils.



Attach the cushion to the mask with the leak correction dial pointing downwards.



Disconnect the headgear by twisting them away from the frame.



With the leak correction dial facing downwards position the cushion over the child's nose.



Pull the headgear over the child's head and press the headgear clips into place to re-attach.



The top of the cushion should rest at the bridge of the nose and cushion should not block the nostrils



Adjust the top and bottom straps to achieve a comfortable fit. Adjust the crown strap and ensure that the frame does not touch the ears.



Connect the Wisp Pediatric tubing and elbow swivel to the front of the mask.

